

Dressing Change for PICC or CVC

REMOVAL

- 01 **Gather appropriate supplies:**
 - Central Line Dressing Change Kit
 - Clean gloves

- 02 **Hand hygiene on entry to room, don clean gloves.**



- 03 **Explain procedure to patient.**

- 04 **Open kit on clean surface and remove glove packet and mask maintaining kit asepsis.**



- 05 **Set gloves aside, off of sterile field.**

- 06 **Don mask.**



- 07 **Remove dressing by pulling towards the insertion site.**



- 08 **Remove Securement device per manufacturer's instructions for use (if applicable) maintaining control of catheter.***



- 09 **Assess site for signs of infection, drainage. Take note of external length.**

- 10 **Remove and discard soiled gloves.**



- 11 **Perform hand hygiene.**




APPLICATION

- 12 **Don sterile gloves.**



- 13 **Cleanse skin with antiseptic solution (CHG)** using a back and forth motion for 30 seconds.***

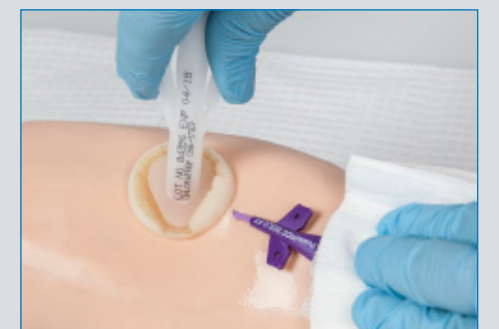

- 14 **Allow to completely air dry.***


- 15 **Apply skin protectant (if applicable) to area larger than Securement device anchor pad, allowing to dry to the touch.***



- 16 **Apply Securement device per manufacturer's instructions for use (if applicable) to :**
 1) PICC/CVC then
 2) patient's skin



- 17 **Apply antimicrobial dressing (if used) following manufacturer's instructions for use.***

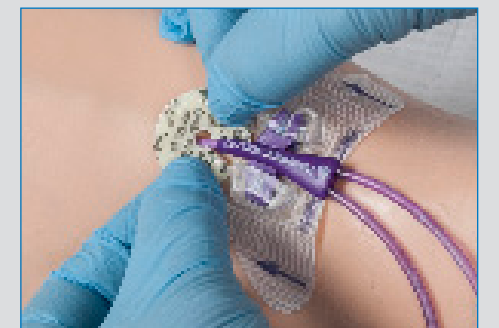

- 18 **Apply dressing to cover the insertion site and catheter bifurcation.**



- 19 **Apply pre-cut securement tape strip under the extension legs and over the dressing border.**



- 20 **Label dressing with date and initials.**



- 21 **Remove gloves, remove mask, and perform hand hygiene.**



- 22 **Document procedure in patient record.**

FOR HOSPITAL CLINICIAN USE ONLY

*Please consult manufacturer's Instructions For Use (IFU) and facility protocols

**Do not use in patients with known allergies to chlorhexidine gluconate or isopropyl alcohol

Note: This document is for informational purposes only and is not intended to be a substitute for clinical judgment and expertise.



BARD ACCESS SYSTEMS, INC. • 605 North 5600 West • Salt Lake City, UT 84116 USA •
 customer service: 800-545-0890 • clinical information: 800-555-7422 • bardaccess.com

Bard, Allpoints, GuardVa, and Statlock are Trademarks and/or Registered Trademarks of C. R. Bard, Inc. All other trademarks are the property of their respective owners. © 2016 C. R. Bard, Inc. All rights reserved. BAS/ALPT/1016/0099

