



BD Nano™ 2nd Gen Pen Needles

Easier injections from start to finish^{2*}


Easier to attach²
Outer cover




Easier to remove²
Inner needle shield



More reliable
4 mm injection depth⁴
Needle base



Easy, comfortable injections^{2,5,6}
4 mm needle



With proven benefits of BD Ultra-Fine™ PentaPoint™ Comfort and EasyFlow™ Technology^{5,6}

Compatible with widely used pen injection devices.⁷

Covered by most health plans at the preferred co-pay, including Medicare Part D.^{||}

^{||}Co-pays and preferred status vary by plan.



BD™ Diabetes Care App
Download today
for recipes, data
logging, how-to-videos
and personalized tips
for self-management.


As part of your injection routine, remember to:

1 Use a new needle.


Pen needles are designed for single use. Using the same needle for multiple injections may be associated with increased injection pain^{3,4†}

2 Rotate injection sites.


Talk to your doctor about an injection site rotation plan that is right for you. The abdomen, thighs, arms and buttocks are generally recommended injection areas. It's important that you rotate your injections within each area and between areas on a regular basis.⁴

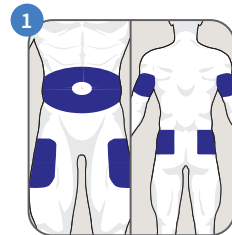


Choose an area.

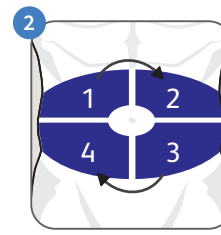

Divide that area into four sections.

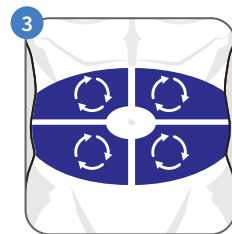

Select an injection site in a section to start injecting. Use one section per week.

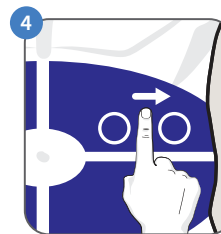

Inject within sites at least one finger width apart from your last injection.


embecta.com

embecta, formerly part of BD. BD is the manufacturer of the advertised products. embecta and the embecta logo are trademarks of Embecta Corp. BD and the BD Logo are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners. © 2022 Embecta Corp. All rights reserved. 2208772974N2GSFB



embecta, formerly part of BD


BD Nano™ 2nd Gen Pen Needles can help you inject with more comfort and confidence.¹

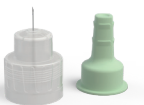


Accept no substitute



Update your injection experience with BD Nano™ 2nd Gen Pen Needles.²

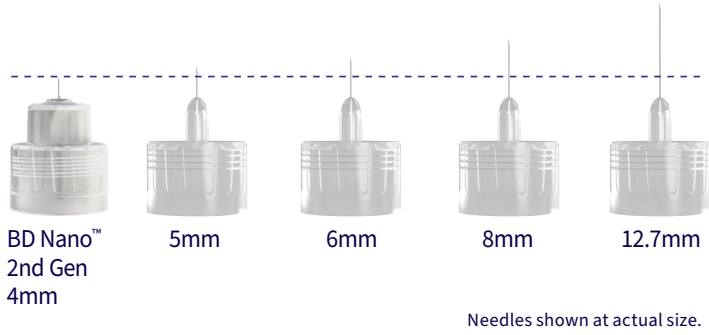


BD Nano™ 2nd Gen Pen Needles – a comfortable contoured needle base that helps you hold the pen against your skin without wobbling, also providing a more reliable injection.^{1,2*†}

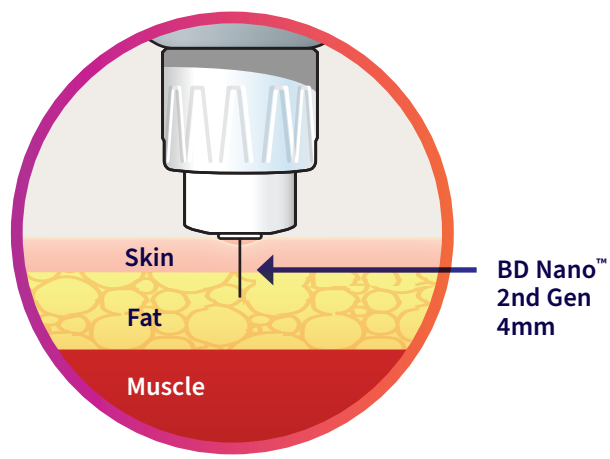

Update your injection experience to a contoured needle base.

Why is it important to inject with shorter needles?

Insulin is meant to be injected into the fat layer **just below the skin** for consistent absorption.³

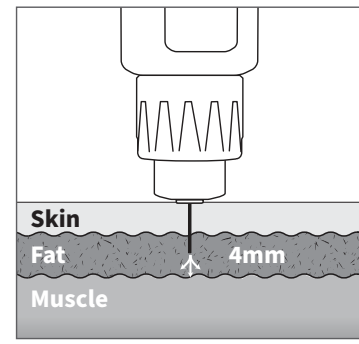

For illustration purposes

Shorter needle lengths may decrease the risk of injecting into the muscle.^{3§}

Each time you refill your insulin, ask your pharmacist if you are due for a refill of your pen needles.

^{*}Compared to other pen needles studied. Results from clinical study assessing patient preferences and feedback. [†]Results from swine studies showing drug under the skin after injection. [‡]Across recommended injection sites: abdomen, thigh, arm, buttock, pooled across gender and BMI. [§]Compared to longer needles.

1. Rini C, Roberts BC, Morel D, et al. Evaluating the Impact of Human Factors and Pen Needle Design on Insulin Pen Injection. J Diabetes Sci Technol. 2019;13(3):533-545. 2. Whooley S, Briskin T, Gibney MA, Blank LR, Berube J, Pflug BK. Evaluating the User Performance and Experience with a Re-Engineered 4 mm×32G Pen Needle: A Randomized Trial with Similar Length/Gauge Needles. Diabetes Ther. 2019;10(2):697-712. 3. American Diabetes Association. Standards of Medical Care in Diabetes – 2022. Diabetes Care. 2022;45(Suppl 1):S1-S264. 4. Frid AH, Kreugel G, Grassi G, et al. New insulin delivery recommendations. Mayo Clin Proc. 2016;91(9):1231-1255. 5. Hirsch L, Gibney M, Berube J, Manocchio J. Impact of a modified needle tip geometry on penetration force as well as acceptability, preference, and perceived pain in subjects with diabetes. J Diabetes Sci Technol. 2012;6(2):328-335. 6. Aronson R, Gibney MA, Oza K, Bérubé J, Kassler-Taub K, Hirsch L. Insulin pen needles: effects of extra-thin wall needle technology on preference, confidence, and other patient ratings. Clin Ther. 2013;35(7):923-933.e4. 7. BD Compatibility Confirmation for Pen Needles and Pen Injector Manufacturers, Document Number: 1490TH-0004-20, Rev S, Dated 28 September 2021.

Try a box for free!

BD Nano™ 2nd Gen Pen Needles 4 mm x 32G
UPC: 382903205509

Offer expires December 31, 2024. Please allow 6 to 8 weeks for rebate processing. ***Only valid for patients with commercial insurance.** See below for details and restrictions.



Manufacturer's mail-in rebate. Expires December 31, 2024. Promotional Code PC2ND

1. Tell your doctor or pharmacist that you are interested in trying **BD Nano™ 2nd Gen Pen Needles** (*prescription is required*).
2. Purchase a box of BD Nano™ 2nd Gen Pen Needles (*100ct products only*) at the pharmacy. **Rebate to cover actual cost to consumer up to a maximum of \$75.**
3. Complete this rebate request and mail along with your valid pharmacy receipt with product name and price circled, and UPC code indicated in this rebate.
4. Mail this form to: **Try Free Box Rebate, P.O. Box 2018, Rock Island, IL 61204-2018.**

Name _____

Address _____

State _____ City _____ Zip _____

Email _____

☐ I would like to receive additional information from embecta and its non-affiliates such as special offers, coupons, product news, surveys, mailings and other communications. By doing so, I understand that the information I am providing may be used by embecta for such business purposes. My information will now be available both to embecta and to such non-affiliates. While embecta will only use such personal data in accordance with our Privacy Statement and Consent, such non-affiliate company's use of your personal data will be subject to the privacy policy of that company and embecta cannot be responsible for their use of your information.

☐ By tendering this rebate request, I certify that: (1) I have read and comply with the terms of this rebate, (2) I am not being reimbursed, nor will I submit a claim for reimbursement, for the purchase of BD insulin syringes or pen needles under any federal healthcare program or state, and (3) I will deduct the amount of the rebate from any claim for reimbursement that I submit to any private insurance program. Offer good only in the United States. Void in MA, VT and elsewhere where prohibited by law, taxed or restricted. Offer not valid for patients with Medicare, Medicaid, VA, TriCare or any other government healthcare program. This offer is not insurance. The selling, purchasing, trading or counterfeiting of this offer is prohibited. This offer is not available to clubs, groups or organizations. Not valid for residents of Puerto Rico. Limit one (1) rebate per household. May not be used with any other discount, coupon or offer. embecta reserves the right to rescind, revoke and/or amend this rebate offer without notice.