

BD Veritor™ System

for Rapid Detection of SARS-CoV2



***** Applicable to customers in the country of Canada only *****

In response to the expanded indications for use issued by Health Canada¹, please refer to these instructions for interpreting test results without the use of the BD Veritor™ Plus Analyzer.

1

Sample collection

Peel down and remove nasal swab

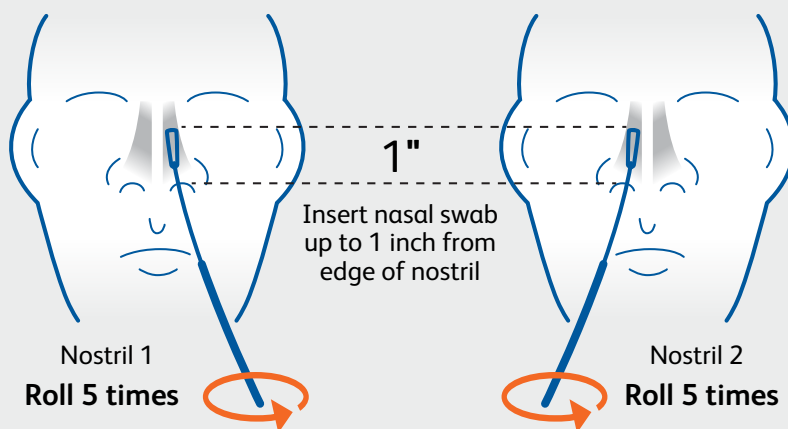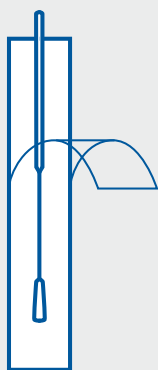


2


Sample preparation



Remove cap from tube and insert nasal swab

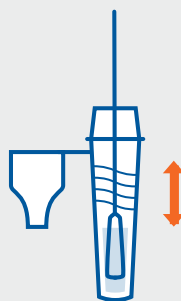

Plunge swab up and down 15 seconds

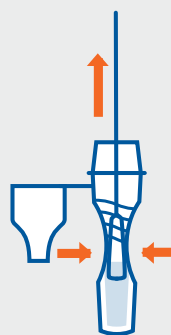

Pinch tube slightly to extract liquid while removing swab




Firmly place cap on tube

Continued on back



3

Sample testing

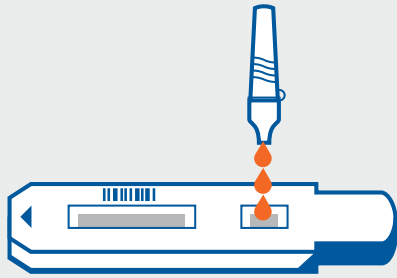

Add 3 drops to the test strip sample well




Wait 15 minutes

4

Result interpretation


Positive result

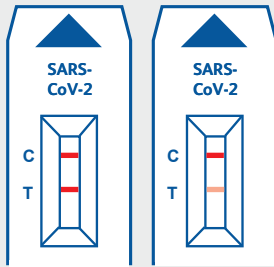

2 lines, one next to C and one next to T, even faint lines, shows the test is **positive**.

Negative result

1 line next to C shows the test is **negative**.

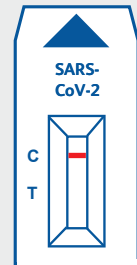


C = Control line
T = Test line

Invalid result


The test is invalid if a non-specific line, a line **not** next to the C or T, is present or if the control line is absent.

Invalid tests should be repeated. If the result is again invalid, the specimen cannot be interpreted.


Test line only

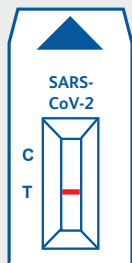


Control line and non-specific line

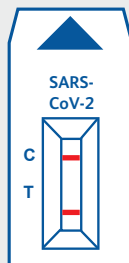


Test line and non-specific line

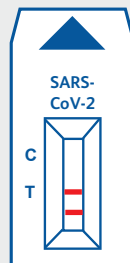


Non-specific line only

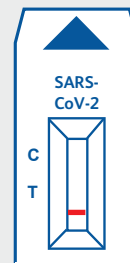

No lines

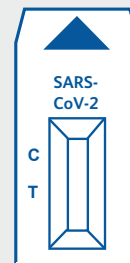

3 lines

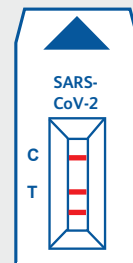

Dos and don'ts of sample collection

- Do collect sample as soon as possible after onset of symptoms.
- Do test sample immediately.
- Use only swabs provided with the kit.
- Refer to: Interim Guidelines for Collecting, Handling and Testing Clinical Specimens from persons for COVID-19 at <https://www.cdc.gov/coronavirus/2019-ncov/lab/guidelines-clinical-specimens.html>

References:

- 1) Health Canada - Expanded Indications for Use (<https://www.canada.ca/en/health-canada/services/drugs-health-products/covid19-industry/medical-devices/authorized/expanded-use.html>)
- 2) BD™ Kit for Rapid Detection of SARS-CoV-2 instructions for use, package insert. Franklin Lakes, NJ: Becton, Dickinson and Company.

- This test has been authorized for sale in Canada by Health Canada under Interim Order.
- This test is only authorized for the duration of the interim order respecting the importation and sale of medical devices for use in relation to COVID-19, unless the authorization is terminated or revoked sooner.

