

Benefits of using the PureWick™ System.



Helps keep the skin dry by wicking away urine.



May help to reduce the risk of nighttime falls, by staying in bed.*



Can make better sleep possible.*



The PureWick™ System is available for home use.



PureWick™ System

Call today for product information or to place an order.

866.587.4748

Indication: The PureWick™ Female External Catheter used with the PureWick™ Urine Collection System is intended for non-invasive urine output management in female patients. **Contraindication:** Do not use the PureWick™ Urine Collection System with PureWick™ Female External Catheters on individuals with urinary retention. **Warnings:** To avoid potential skin injury, never push or pull the PureWick™ Female External Catheter against the skin during placement or removal. Never insert the PureWick™ Female External Catheter into the vagina, anal canal or other body cavities. Do not use the PureWick™ Female External Catheter with a bedpan or any material that does not allow for sufficient airflow. Discontinue use if an allergic reaction occurs. Not recommended for users who are experiencing skin irritation or skin breakdown in device contact areas. Please consult package insert for more detailed safety information and instructions for use.

C. R. Bard, Inc.
Covington, GA, 30014, U.S.

BD, the BD Logo and PureWick are trademarks of Becton, Dickinson and Company and its affiliates.
© 2019 BD. All rights reserved.
2004-18 BD-14942

PureWick™ System

Available for home use.

A breakthrough for women with urinary incontinence.

Over 2,000 hospitals and rehab facilities are using the PureWick™ Female External Catheter.

*Based on survey responses (01/19) by 141 US caregivers and users who purchased the PureWick™ System and a subsequent supply of PureWick™ Female External Catheters, in each case solely for the homecare setting.



What is the PureWick™ System?

The PureWick™ Urine Collection System is designed to manage your urinary incontinence. It works with the PureWick™ Female External Catheter to wick urine away from the body into a designated collection canister. You can benefit from the PureWick™ Urine Collection System in the home.

Frequently asked questions.

1. Will I feel the suction used with the PureWick™ Urine Collection System?

The PureWick™ Urine Collection System is designed so that suction should not be felt. The device is held in place by your anatomy.

2. How often should the PureWick™ Female External Catheter be changed?




The device should be changed at least every 8-12 hours, or if soiled with feces or blood.

3. Can the PureWick™ Urine Collection System be used when the patient is on her side or sitting in a chair?


Yes – the PureWick™ System may be used when the patient is lying or reclining on her back or side, as well as when seated in a chair. Care must be taken to ensure that the PureWick™ Female External Catheter is not positioned beneath the tailbone.

PureWick™ System

How does it work?

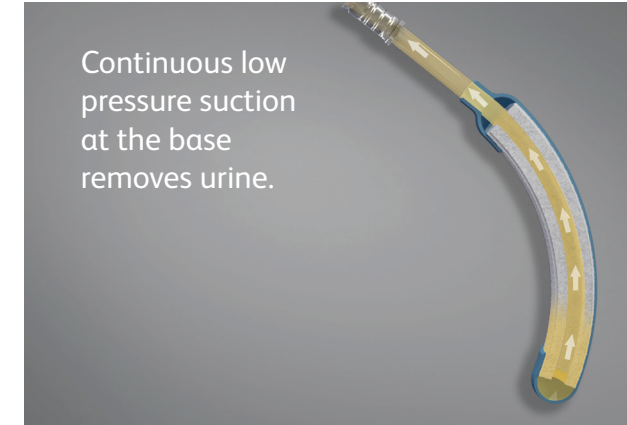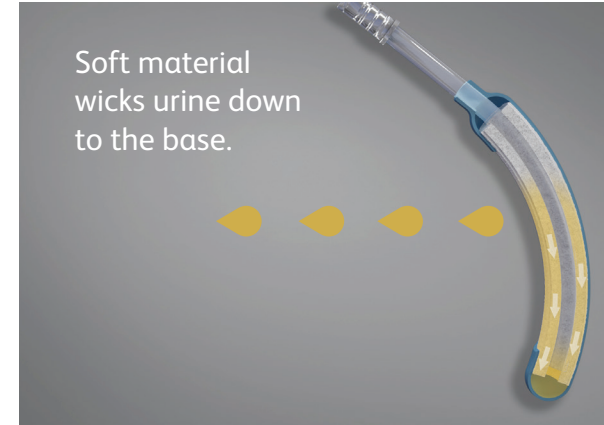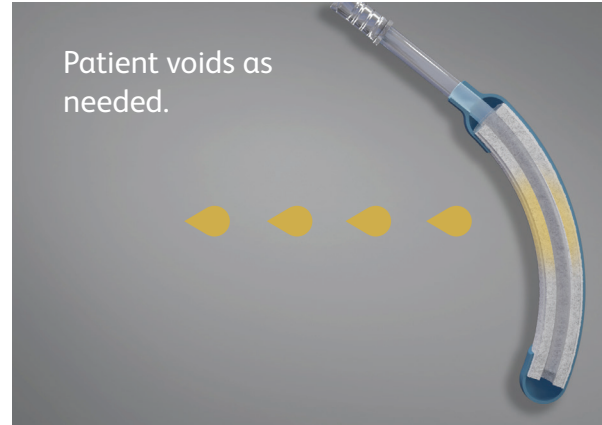

The PureWick™ Female External Catheter connects to the PureWick™ Urine Collection System.

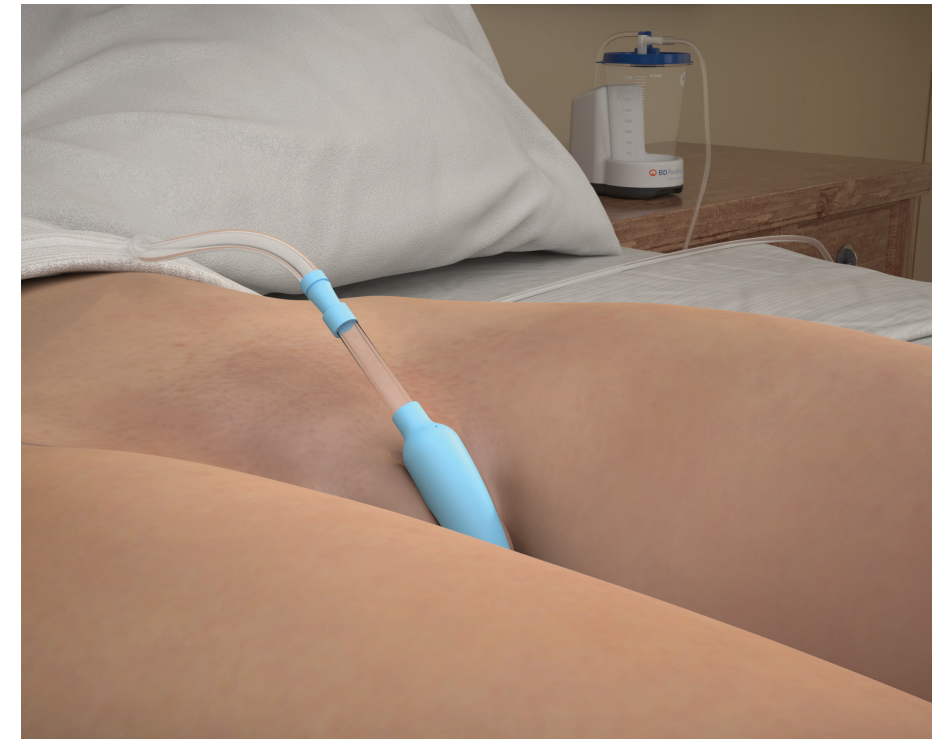

The PureWick™ Female External Catheter is placed between the legs in order to collect urine from the user.